Manuale d'Uso

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User Manuel

Dynamic Flow is flowable, light-cured, radiopaque lowable composite resin for invasive cavity preparation.

Indication: Restoration of invasive cavity Pit and fissure sealing Base and Liner for direct restoration Repair acrylic temporary materials Repair of esthetic indirect restorations

Contra-Indications: Dynamic Flow should not be used if patients are known to be allergic to methacrylate resins.

Instructions Before isolating and preparing choose the desired shade and type of Dynamic Flow. Clean with a slurry of pumice and use and dry. Apply 37% PD etching gel to avabile enamel dentin 15 sec., remove gel and rinse for 20 sec. Blot dry, keep moist and do not rub. Apply bond to the tooth. Apply double coat of President Prebond by using microbrush. Air dry gently for 5 sec. Light Cure for 10 sec. If you apply other products, use them according to manufacturers instructions. Dispensing-Dynamic Flowable composite can be delivered directly from the disposable tip. Place and light cure restorative in increments. Cure

each increment by its entire surface to curing light for 20 seconds. Finish and Polish-After contouring composite, polish surface with finishing and polishing proximal areas. Occlusion-Check lateral and centric occlusion adjust as necessary.

Cautions: Avoid contact of Dynamic Flow with the skin/mucous membrane or eyes. None polymerized Dynamic Flow may causes lightly, reversible irritation or lead to a general sensitization against methacrylates. Keep away from children. For dental use only. Do not use after the date of expiration. Exposure to light may lead to premature polymerization.

Storage Store at 1-30°C. After use close syringe cap. Avoid from direct sun light.

Shelf Life 3 years after manufacturing date

Components Kit Content Dynamic Flow Kit 5x2g syringes Shade: A2, A3, A3.5, B2,B3 Dynamic Flow Refill 2x 2g syringes Shade: A1, A2, A3, A3.5, A4, B1, B2, B3, A2O, A3O, C2, D2, WT, TL

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