Presigum

Hydrophilic Addition Silicone
A- Silicone

www.presidentdental.de

DATE OF ISSUE 15.02.2020

REFERENCE CODE

PRD.01.PGUML01

PRD.01.PGUML02

PRD.01.PGUMP01

PRD.01.PGUMP02

PRESIDENT DENTAL
STUDY CLUB

*Follow President Dental Product Clinical Cases On

 President Dental Study Club*

INSTRUCTIONS FOR USE

Presigum Putty Soft: Vinylpolysiloxane for first impressions with elastic final consistency, recommended for all impression techniques: two-stage (double impression*) or single-stage (simultaneous).

Presigum Tray Material: High viscosity vinylpolysiloxane. Recommended for filling impression trays in conjunction with a 'wash' within the Presigum range (i.e. light body, extra light body & regular body) in the single-stage (simultaneous) technique.

Presigum Monophase: Medium viscosity hydrophilic vinylpolysiloxane. A single viscosity monophase material recommended for use as a preparation wash and for filling individual impression trays. Also if desired this Monophase can be used in conjunction with a 'wash' from the Presigum range (i.e. light body, extra light body & regular body) in the single-stage (simultaneous) technique.

Presigum Regular Body: Medium viscosity hydrophilic vinylpolysiloxane. Recommended for use in conjunction with Presigum Putty or Tray Material in the single-stage (simultaneous) technique, or as a monophase material in a special tray.

Presigum Light Body: Low viscosity hydrophilic vinylpolysiloxane. Recommended for use in conjunction with Presigum Putty or Tray Material in the two-stage (double impression*) or single-stage (simultaneous) technique.

** When using this product in the two-stage (double impression) technique, we recommend that you place a polythene spacer sheet on the putty during the first stage of impression taking to improve detail quality in the next stage.*

IMPRESSION TRAYS: PREPARATION AND ADHESIVES

All standard and/or special trays can be used, provided they are rigid and non-deformable. Proper adhesion of the material to the impression tray is essential to avoid errors, especially during removal from the mouth. To ensure a strong bond, make sure that the adhesive is appropriate for the type and material of the tray.

IMPORTANT: When using special trays, ask the technician if the resin used is self or light curing. For correct use, follow the instructions for each individual adhesive.

INSTRUCTIONS FOR USE

Presigum Putty Mixing:

- Take the 2 components out of the tubs using the colour coded dosing spoons.
- Mix identical proportions of Base and Catalyst; until you obtain a homogeneous mix, with no streaks (about 30 sec.) Then place into the impression tray.

WARNINGS:

- Proportions must be strictly observed in order to have the right setting times: a larger quantity of catalyst does not accelerate the setting time.
- Close tubs immediately after use. Do not invert lids, lower lids and dosing spoons.
- Contact with latex gloves may prevent polysiloxanes from hardening. Avoid direct or indirect contact with latex gloves. Instead, wash your hands well to eliminate all traces of impurities or use vinyl gloves.
- Take care not to stain your clothing as vinyl polysiloxanes are chemically resistant.

PRESIGUM FLUIDS CARTRIDGE VERSION PREPARING THE DEVICE:

Warning: Before assembling the tip, ensure that the two components (base and catalyst) flow out evenly by applying a light pressure on the dispenser lever and extruding a small amount of material, which must be removed. Subsequently, insert the mixing tip and, if required, the intraoral tip in the cartridge.

PRESIGUM FLUIDS TUBE VERSION

(Only for Presigum Light Normal and Presigum Regular)

Mixing: Spread the components in lines of equal length on the mixing pad. Mix thoroughly with the spatula, until a homogeneous colour is obtained, according to the times shown on the schedule. After use, recap the tube, making sure the caps are not reversed.

WARNING: In sensitive subjects, vinyl polysiloxanes may cause irritation or other allergic reactions.

CLEANING AND DISINFECTION OF THE IMPRESSION

After rinsing well under running water, the impression can be immediately disinfected.

INSTRUCTIONS FOR USE

CASTING MODELS

There is no need to wait; models may be cast immediately after disinfection, or up to two weeks later, provided that the impression is kept at room temperature. Presigum is compatible with all the best plasters on the market and with polyurethane resins..

CLEANING AND DISINFECTING THE DISPENSER

The dispenser can be cleaned, disinfected, and sterilised by immersion in liquid or in a steam autoclave.

STORAGE

Impressions made with Presigum must be kept at room temperature. Presigum impression materials are guaranteed for 3 years following the date of production provided they are kept at a temperature of between 5° and 27°C / 41°- 80°F.

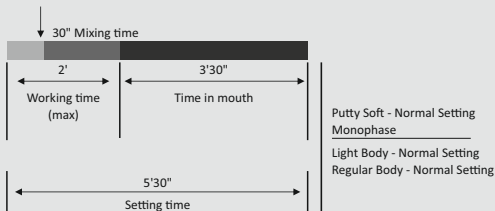
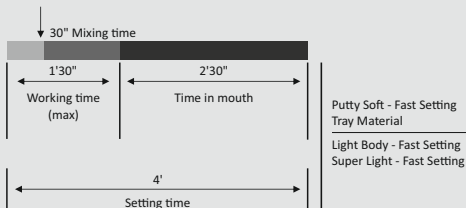
GALVANISATION

Presigum impressions may be galvanised with copper or silver.

IMPORTANT REMARKS: Consulting services on the use of our products, provided in any form, be that verbal, written or through demonstrations, are based upon the current state of dentistry and of our know-how. They are to be considered as non-binding information, also in relation to possible third party claims, and do not exonerate the user from personally checking whether the product is suitable for the planned use. Use and application by the user take place under no possible control by the company, and are therefore under the user's responsibility. Possible liability for damages is limited to the value of goods supplied by the company and utilized by the user.

INSTRUCTIONS FOR USE

Time For Clinical Use

INSTRUCTIONS FOR USE

TECHNICAL DATA

Presigum	Light Body Normal Setting	Reg. Body Normal Setting	Light Body L.Fast Setting	Monophase	Tray Material	Putty Soft Normal Setting	Putty Soft Fast Setting
ISO 4823	Type 3 Light-bodied Consistency	Type 2 Med.-bodied Consistency	Type 3 Light-bodied Consistency	Type 2 Med.-bodied Consistency	Type 1 Heavy-bodied Consistency	Type 0 Putty-bodied Consistency	Type 0 Putty-bodied Consistency
Mixing ratio Base: Catalyst	1:1	1:1	1:1	1:1	1:1	1:1	1:1
Mixing Time	in tube 30"	in tube 30"	Auto mixing system	Auto mixing system	Auto mixing system	30"	30"
Total Working Time*	2'	2'	1'30"	2'	1'30"	2'	1'30"
Time in Mounth (min.)	3'30"	3'30"	2'30"	3'30"	2'30"	3'30"	2'30"
Setting Time*	5'30"	5'30"	4'	5'30"	4'	5'30"	4'
Strain in Compression (min - max)	3-5%	3-5%	3-5%	3-5%	3-5%	1-3%	1-3%
Recovery From Deformation	>99,5%	>99,5%	>99,5%	>99,5%	>99,5%	>99,0%	>99,0%
Linear Dimen- sional change 24 hours	<0,20%	<0,20%	<0,20%	<0,20%	<0,20%	<0,20%	<0,20%
Hardness "Shore A"	45±2	45±2	45±2	54±2	60±2	60±2	60±2

The times mentioned must be intended from the start of mixing at 23°C - 73°F. Higher temperatures reduce the times, lower temperatures increase them. **"FOR DENTAL USE ONLY"**.

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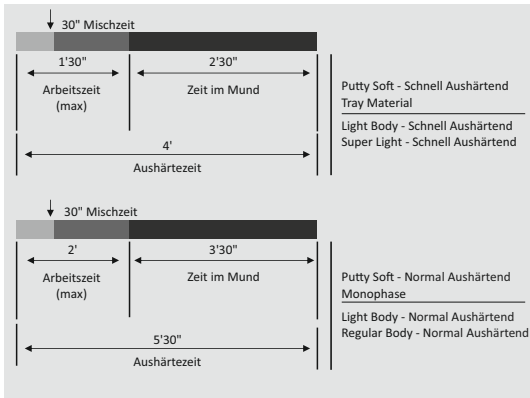
Presigum	Light Body Normal Setting	Reg. Body Normal Setting	Light Body L.Fast Setting	Monophase	Tray Material	Puty Soft Normal Setting	Puty Soft Fast Setting
ISO 4823	Type 3 Light-bodied Consistency	Type 2 Med.-bodied Consistency	Type 3 Light-bodied Consistency	Type 2 Med.-bodied Consistency	Type 1 Heavy-bodied Consistency	Type 0 Putty-bodied Consistency	Type 0 Putty-bodied Consistency
Dosierung Base: Katalysator	1:1	1:1	1:1	1:1	1:1	1:1	1:1
Anmischzeit	in tube 30"	in tube 30"	Auto mixing system	Auto mixing system	Auto mixing system	30"	30"
Gesamtverar- beitungszeit*	2'	2'	1'30"	2'	1'30"	2'	1'30"
Verweildauer im Mund (min.)	3'30"	3'30"	2'30"	3'30"	2'30"	3'30"	2'30"
Abbindezeit*	5'30"	5'30"	4'	5'30"	4'	5'30"	4'
Verformung unter Druck (min - max)	3-5%	3-5%	3-5%	3-5%	3-5%	1-3%	1-3%
Rückstellung nach Verformung	>99,5%	>99,5%	>99,5%	>99,5%	>99,5%	>99,0%	>99,0%
Lineare Dimensionsab- weichung nach 24 Stunden	<0,20%	<0,20%	<0,20%	<0,20%	<0,20%	<0,20%	<0,20%
Härte "Shore A"	45±2	45±2	45±2	54±2	60±2	60±2	60±2

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Puty Soft Fast Setting	Puty Soft Normal Setting	Tray Material	Monophase	Light Body L.Fast Setting	Reg. Body Normal Setting	Light Body Normal Setting	Presigum
Type 0 Putty-bodied Consistency	Type 0 Putty-bodied Consistency	Type 1 Heavy-bodied Consistency	Type 2 Med.-bodied Consistency	Type 3 Light-bodied Consistency	Type 2 Med.-bodied Consistency	Type 3 Light-bodied Consistency	ADA 19
1:1	1:1	1:1	1:1	1:1	1:1	1:1	نسبة طلخا بمسانلا: المادة فز فحلما
30"	30"	Auto mixing system	Auto mixing system	Auto mixing system	in tube 30"	in tube 30"	مدة طلخا
1'30"	2'	1'30"	2'	1'30"	2'	2'	مدة ريضحتنا للاجمالية *
2'30"	3'30"	2'30"	3'30"	2'30"	3'30"	3'30"	قبته ففسية قألحا (قأنألال)
4'	5'30"	4'	5'30"	4'	5'30"	5'30"	مدة بصلمتا *
1-3%	1-3%	3-5%	3-5%	3-5%	3-5%	3-5%	داسطا عند طلغضا (ده بندال ده بصفاق)
>99,0%	>99,0%	>99,5%	>99,5%	>99,5%	>99,5%	>99,5%	ونقره ملاأعاده
<0,20%	<0,20%	<0,20%	<0,20%	<0,20%	<0,20%	<0,20%	لأناأبعاد 24 نأأعالم
60±2	60±2	60±2	54±2	45±2	45±2	45±2	بأأصلا Shore A

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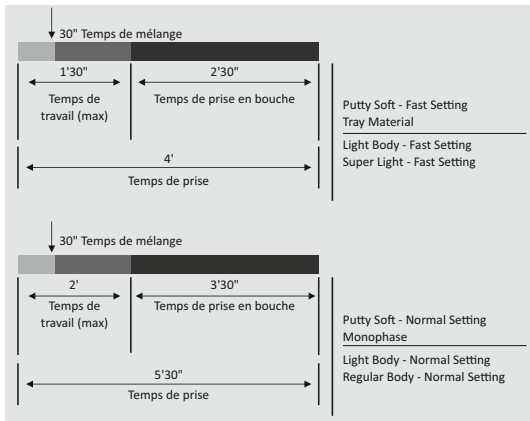
Presigum	Light Body Normal Setting	Reg. Body Normal Setting	Light Body L.Fast Setting	Monophase	Tray Material	Puty Soft Normal Setting	Puty Soft Fast Setting
ISO 4823	Type 3 Light-bodied Consistency	Type 2 Med.-bodied Consistency	Type 3 Light-bodied Consistency	Type 2 Med.-bodied Consistency	Type 1 Heavy-bodied Consistency	Type 0 Putty-bodied Consistency	Type 0 Putty-bodied Consistency
Dosage Base: Catalyseur	1:1	1:1	1:1	1:1	1:1	1:1	1:1
Temps de mélange	in tube 30"	in tube 30"	Auto mixing system	Auto mixing system	Auto mixing system	30"	30"
Temps total de travail*	2'	2'	1'30"	2'	1'30"	2'	1'30"
Temps de prise en bouche (min)	3'30"	3'30"	2'30"	3'30"	2'30"	3'30"	2'30"
Temps de prise*	5'30"	5'30"	4'	5'30"	4'	5'30"	4'
Déformation sous compression (min - max)	3-5%	3-5%	3-5%	3-5%	3-5%	1-3%	1-3%
Mémoire élastique	>99,5%	>99,5%	>99,5%	>99,5%	>99,5%	>99,0%	>99,0%
Variation dimensionnelle linéaire après 24 heures	<0,20%	<0,20%	<0,20%	<0,20%	<0,20%	<0,20%	<0,20%
Dureté " "Shore A"	45±2	45±2	45±2	54±2	60±2	60±2	60±2

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