

# NOVESEAL®

## User Manual

Light-curing, radiopaque, single component cavity liner.

### Application

Prepare the cavity, clean it thoroughly and dry well. Place the required amount of material on a mixing pad shortly before applying it. Avoid exposure to daylight or to the operating light as this can cause the material to harden prematurely.

Apply a layer of the lining material with a small round condenser. The layer should not exceed 1 mm in thickness. Should a thicker lining be necessary, apply and cure several individual layers. The material should be applied to dentine only. Do not apply it to enamel.

The material is cured by using light curing. The light guide should be positioned as close as possible to the cavity liner. When the light guide is 2 mm above the surface, cure the liner for at least 30 seconds. Under no circumstances cure for less than 20 seconds.

Possible tenacious remains of the material on the enamel should be removed by using a rotary instrument.

### Additional Information

The liner is so strong after curing that a probe cannot be used to remove excess material from cavity linings. This would loosen the entire lining. Instead, use a rotary instrument to carefully remove the excess. The material should be stored at room temperature, but not exceeding 20°C. Refrigerating the material (4-8°C) will increase its shelf-life. Allow the material to resume room temperature before use.

### Side effects

Undesirable side-effects to this medical-product are not to expect, if working corresponding to the instructions of use. In case of side-effects - even when doubtful - please inform us with exact description circumstances and symptoms.

### Contra-indication

Not for use in patients who are known to be allergic to derivatives of methacrylic acid.

### Interaction with other materials

The liner should not be used in conjunction with materials containing eugenol, as eugenol impedes its setting reaction.

### Instruction references

Take care of all working- and caution advices. In case of unexpected side effects or other reactions (even if work was made according to the Instructions of use) please inform us, if possible, with

exact description of the accompanying circumstances and symptoms.

### NOTE

Applicator-tools (tips, brushes) are only determinate for single-use at only one patient.

### Presentation

REF PRD.333.01.02 1 x 2,5 g

REF PRD.333.02.02 2 x 2,5 g



President Dental GmbH

Zehentstadelweg 7,81247  
München /GERMANY

Tel: +49 89 127660240  
Fax +49 89 127660269

www.presidentdental.de  
info@presidentdental.de

Date of issue 2014.02.01 IFU02E(Rev.2)