

Nova-Tray



Description

Nova-Tray is a transparent light-curing individual impression tray resin. It is easy and convenient to handle than chemical curing tray resin because it can be used without mixing in a direct way unpleasant smell.

Composition

- Urethane dimethacrylate (UDMA)
- Glass
- Photo-initiators
- Pigment

Indications

1. Individual tray for use in crown, bridge and total prosthetic technique.
2. Base plates for bite patterns and odontograms

Indications

1. Mark margins on the gypsum model for individual tray. Use a space maintainer such as wax sheet if necessary
2. Fit the Nova-Tray on the gypsum model.
3. Cut the excess materials along a boundary line. To make a handle can be removed materials
4. Put the gypsum model with tray resin into a UV light curing unit and polymerize for 3 to 5 minutes. Then remove the tray resin from the gypsum model, turn it over and expose the reverse side to the light. A longer light exposure can improve the physical properties of the material.

Storage

1. Dry and store in a cool place
2. Do not store at elevated temperature or intense light.
3. Put the lid back on after using it.

Precautions

1. We only for dental
2. Only dentist and dental professional can be used it. Observe the procedure and the precautions.
3. Keep it away from a reach of non-dental users including children, senior citizens and others.
4. Do not use Nova-Tray to a patient having inflammation or allergy against acrylate.
5. Exposure to uncured resin should be avoided. Put on the protective gloves using Nova-Tray.
6. If contacts skin, wash immediately with soap and water.
7. If accidental contact with eyes or prolonged contact with oral tissues occurs, flush immediately with a large amount of water.
8. Do not use after expiry date.

Symbols



Warranty

President Dental Handels GmbH will replace product that is proven to be defective. President Dental Handels GmbH does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, the user should determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.



President Dental GmbH zehenstadelweg 7
81247 München, Germany Tel: 0049 89 127 660 240
www.presidentdental.com