NOWAX

Dental Modelling Wax

The popular properties of dental modelling wax should be: -

- 1) High strength and rigidity at mouth temperature.
- 2) Wide softening range above mouth temperature.
- 3) Easily mouldable in the softened state, without flaking, cracking or tearing.
- 4) There should be low thermal contraction.
- 5) Easily carvable at room temperature without flaking or chipping.
- 6) There should be little change in properties on melting and re-solidification.
- 7) There should be no residue on boiling out.

Direction of use

Take a sheet of wax and tear in half, soften the sheet of wax over a Bunsen flame and press it on to the plaster model. Make sure the wax goes into all the curves and cracks on the model and then trim/carve the excess wax away using a heated knife.

Take a sheet of wax and soften over a Bunsen flame and mould the wax into a block, this would be suitable for bite registration.

Note:

Baseplate waxes shall be of such texture that they carve cleanly and sharply at room temperature using normal dental carvers and normal dental laboratory technique.

Softening: - the wax should soften without becoming flaky or crumbly and without adhering to the fingers.

Trimming/ Carving: - the wax should trim easily and clearly without tearing or chipping.

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