

Prophy Powder®

Prophylaxis Powder



President Dental® Prophy Powder is a spearmint flavored sodium bicarbonate powder for professional air polishing prophylaxis procedures. In conjunction with an air polishing system, Prophy Jet Powder allows removal of stains such as tobacco, coffee, tea. It is also indicated for prophylaxis of orthodontic patients as well as preparation of tooth surfaces prior to bonding and sealant procedures.

Composition

The product contains: Sodium Bicarbonate

Product Presentation

600gm Bottle
4.5kgs Container


Instructions for use:

1. Shake well before using.
2. Use according to polishing unit manufacturer's instructions.

Warning

1. Use is not recommended for patients with known allergies to the components.
2. Do not direct the air polishing stream at soft tissue or into the sulcus.
3. Do not modify powder with additives.

Use and Storage

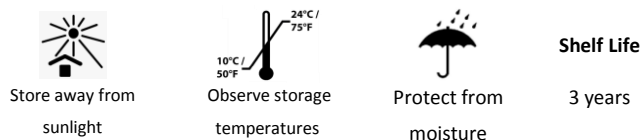

Emergency Measures

In case of accidental contact, rinse with plenty of water. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

Präsident Dental® Prophy Powder ist ein grüner Minze gewürzt Natriumbicarbonatpulver für professionelle Luft Polieren Prophylaxe Verfahren. In Verbindung mit einem Luftpoliersystem, Prophy Jet ermöglicht Powder Entfernung von Flecken wie Tabak, Kaffee, Tee. Es ist auch für die Prophylaxe von orthodontischen Patienten sowie der Vorbereitung von Zahnoberflächen vor der Verklebung und Versiegelung Verfahren angegeben..

Zusammensetzung

Das Produkt enthält: Natriumbikarbonat

Produktvorstellung

600gm Bottle
4.5kgs Container

Gebrauchsanweisung

1. Gut schütteln vor dem Gebrauch.
2. Verwendung nach einem der Poliereinheit Herstellers.

Warning

1. Die Nutzung ist nicht für Patienten mit bekannter Allergie zu den Komponenten empfohlen.
2. Setzen Sie den Luftpolierstrom bei Weichgewebe oder in den Sulkus zu lenken.
3. Nicht Pulver mit Zusatzstoffen zu ändern.

Use und Speicherung



Emergency Measures

Bei versehentlichem Kontakt gründlich mit Wasser abspülen. Bei Berührung mit den Augen sofort gründlich mit Wasser abspülen und Arzt konsultieren.



Zehentstadelweg 7 81247 München / Germany
Tel: 089-127 660 240 Fax: 089-127 660 269
www.presidentdental.com info@presidentdental.com

made in germany

Polishing Powder - Polierpulver

